PRECIPIT

One-Step Surface Cleaner and Disinfectant Wipes formulated with ACCELERATED HYDROGEN PEROXIDE



WHY CHOOSE PREempt™?

- **CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.
- **FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.
- **RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds) and PPE** is not required as per health and safety rating.
- **SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residues which helps to reduce environmental impact.

DIRECTIONS FOR USE

Disinfection of non-porous hard surfaces such as the exterior of fume hoods, tabletops and other environmental surfaces.

- 1. Heavily soiled surfaces require cleaning prior to disinfection.
- 2. Apply to surface with disposable wipe.
- **3.** Ensure surface remains wet for 3 minutes.

Disinfection of Non-Critical, Pre-cleaned Instruments and Medical Devices:

- Instruments must be thoroughly pre-cleaned to remove excess organic debris, rinsed, and dried.
- 2. Wipe the instrument with PREempt™ Wipes and allow surface to remain wet for 3 minutes.
- 3. Wipe with a clean, damp cloth or paper towel and allow to air dry.

For use in laboratories, clean rooms and other critical environments that require cleaning and surface disinfection. This includes work stations, fume hoods, laboratory counter tops, equipment and other hard non-porous environmental surfaces.

AVAILABLE PRODUCTS

PREempt™ One-Step Surface Cleaner and Disinfectant Wipes

DIN: 02436671

SHELF LIFE: 2 Years

CONCENTRATION OF ACTIVE: 0.5% Hydrogen Peroxide

• **Format Size:** 6"x7" – 160ct Wipes

Product Code: 11221

Packaging Specifications: 12x1 Case

• **Format Size:** 7.5"x10" – 10ct Wipes (Soft

Pack)

Product Code: 11223

Packaging Specifications: 1 Case x 6 inner

Cartons x 20 Soft Packs



EFFECTIVE AGAINST A BROAD-SPECTRUM OF BACTERIA, VIRUSES AND FUNGI IN 3 MINUTES!

Virucidal: 3 Min.

- Proven effectiveness against the Poliovirus Type 1, Sabin strain type 1 which allows for a Broad-Spectrum Virucidal claim against most enveloped and non-enveloped viruses
- HIV-1 Human Immunodeficiency Virus (HIV)
- Human Coronavirus 229E
- Feline Calicivirus, F9 Strain, as a surrogate for Norwalk and Norwalk-like viruses
- This product has demonstrated effectiveness against Poliovirus and is expected to inactivate all Influenza A viruses including 2009 (H1N1) pandemic Influenza A virus

Bactericidal: 3 Min.

- · Staphylococcus aureus
- Pseudomonas aeruginosa
- Salmonella enterica
- Methicillin-resistant Staphylococcus Aureus (MRSA)
- Vancomycin-resistant Enterococcus faecalis (VRE)

Fungicidal: 3 Min.

• Trichophyton mentagrophytes

Tuberculocidal: 3 Min.

• Mycobacterium terrae

Broad-Spectrum Sanitizing: 30 Sec.

- Staphylococcus aureus
- Pseudomonas aeruginosa
- Salmonella enterica
- Klebsiella pneumoniae
- Escherichia coli O157:H7
- Campylobacter jejuni



PREempt™ One-Step Surface
Cleaner and Disinfectant Wipes
are non-woven, meltblown
polypropylene wipe material,
ensuring even surface coverage
and increasing the ability and
efficiency of AHP® for fine particle
removal.

Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

 $\textit{PREempt}^{\text{\tiny{TM}}} \ branded \ products \ are \ manufactured \\ \ by \ Virox \ Technologies \ Inc.$

Accelerated Hydrogen Peroxide® (AHP®) and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:



