PREempt

Concentrated One-Step
Surface Cleaner and
Disinfectant formulated
with ACCELERATED*
HYDROGEN PEROXIDE
Patented



WHY CHOOSE PREempt™?

- **CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.
- **FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.
- **RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds) and PPE** is not required for in use solutions as per health & safety rating.
- **SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residues which helps to reduce environmental impact.

DIRECTIONS FOR USE

Use Dilutions Made Easy

RATIO	PER 1 LITRE OF WATER	PER 4 LITRES OF WATER
1:40	25mL	100mL
1:128	8mL	32mL

Disinfection of non-porous hard surfaces coming in contact with intact skin such as the exterior of fume hoods, tabletops and other environmental surfaces.

- 1. Dilute 1:40.
- 2. Apply to surface with cloth or disposable wipe.
- **3.** Ensure surface remains wet for 5 minutes.

For use in laboratories,
clean rooms and other
critical environments that
require cleaning and surface
disinfection. This includes work
stations, fume hoods, laboratory
counter tops, equipment
and other hard non-porous
environmental surfaces



AVAILABLE PRODUCTS

PREempt[™] Concentrated One-Step Surface Cleaner and Disinfectant

DIN: 02436701

SHELF LIFE: 2 Years, 30 Days Diluted
CONCENTRATION OF ACTIVE: 7% Hydrogen
Peroxide (Concentrated)

Format Size: 4 L Bottle
 Product Code: 11305
 Packaging Specifications: 4x4 L

• Format Size: 200 L Drum Product Code: 11311

Packaging Specifications: $1\times200\ L$



Disinfection of Non-Critical, Pre-cleaned Instruments and Devices:

- 1. Dilute to 1:40.
- 2. Instruments must be thoroughly pre-cleaned to remove excess organic debris, rinsed, and dried.
- **3.** Thoroughly clean and rinse lumens of hollow instruments.
- **4.** Using either a tray or an ultrasonic unit, immerse instrument in use dilution for 5 minutes.
- **5.** Remove and rinse instruments.
- 6. Wipe dry prior to use.
- 7. Discard solution after each use.

EFFECTIVE AGAINST A BROAD-SPECTRUM OF VIRUSES, BACTERIA, FUNGI AND MYCOBACTERIA IN 5 MINUTES!

Virucidal: 5 Min. (1:40 dilution)

- Poliovirus Type 1, Sabin strain type 1 which allows for a Broad-Spectrum Virucide claim against most enveloped and non-enveloped viruses
- HIV-1 Human Immunodeficiency Virus (HIV)
- Human Coronavirus 229E
- Feline Calicivirus, F9 Strain, as a surrogate for Norwalk and Norwalk-like viruses
- Effective against Poliovirus and is expected to inactivate all Influenza A viruses including 2009 (H1N1) pandemic Influenza A virus

Bactericidal: 5 Min. (1:40 dilution)

- Staphylococcus aureus
- Pseudomonas aeruginosa
- Salmonella enterica
- Methicillin-resistant Staphylococcus Aureus (MRSA)
- Vancomycin-resistant Enterococcus faecalis (VRE)

Fungicidal: 5 Min. (1:40 dilution)

• Trichophyton mentagrophytes

Tuberculocidal: 5 Min. (1:40 dilution)

Mycobacterium terrae

Broad-Spectrum Sanitizing: 30 Sec. (1:128 dilution)

- Staphylococcus aureus
- Pseudomonas aeruginosa
- Salmonella enterica
- Klebsiella pneumoniae
- Escherichia coli O157:H7
- Campylobacter jejuni



PREempt™ Concentrate

One-Step Surface Cleaner and

Disinfectant In-Use Solutions

were designed to be easier on

staff and occupants with half

the contact time compared

to traditional Quaterenary

Ammonium Compound

products.

Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

 $PREempt^{\mathsf{TM}}$ branded products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® (AHP®) and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:



