Instructions Instruction Instrucciones

- 1 Inside your Black Knight GLO package, you will find the Black Knight GLO mat, as well as the GLO light in a small bag. Remove the GLO light from the bag and remove the white plastic battery protection strip from the light by pulling the protection strip gently away from the GLO light.
- 2 Insert the GLO light into the round hole in the mat with the flat side facing up and the rounded portion facing down.
- 3 To activate the GLO light gently push on the center of the light until the light illuminates.
- 4 To turn off the GLO light, gently push on the center of the light until the light turns off.
Please note light is designed for temporary usage. Prolonged usage will cause reduce battery life.
- 5 To replace button battery on the GLO light, simply push the light compartment out of the mat.
- 6 Open the light compartment by prying open the two halves at the top lip of the compartment.
- 7 Gently remove light portion to better access battery.
- 8 Remove the old battery from the slot using a thin blunt instrument. Insert the new battery ensuring that the positive (+) side of the battery is facing up.
- 9 Re-insert light portion into compartment ensuring flat portion is facing down, and that the battery is facing up.
- 10 Replace the cap of the light compartment, and insert back into the mat, again making sure that the rounded side of the light is facing downwards.

*Black Knight GLO uses battery CR2032, 3V Lithium Coin battery.

WARNING: Keep battery* away from small children. If accidentally swallowed, call your doctor immediately.

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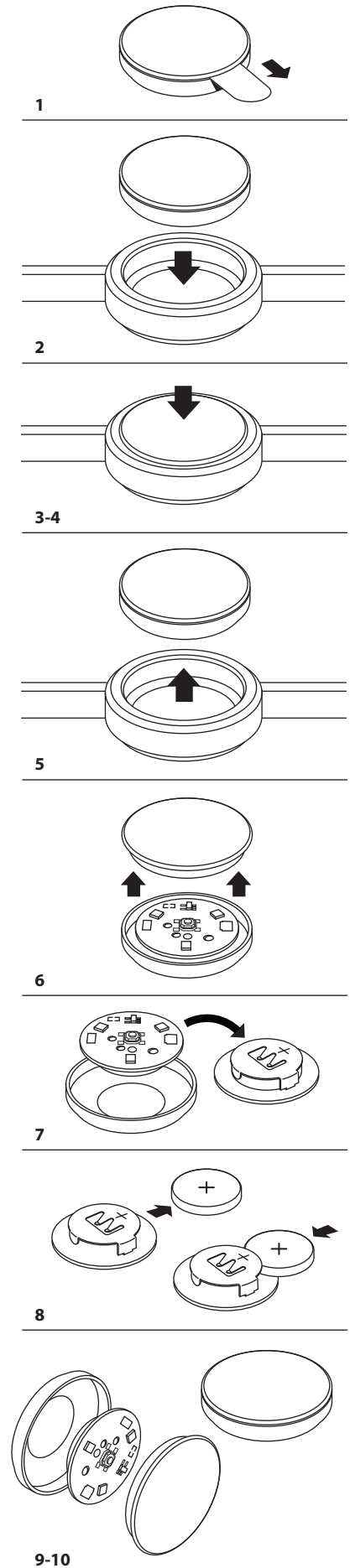
info@cpapology.com
www.cpapology.com

CPAPology® is a Brand of KEGO Corporation